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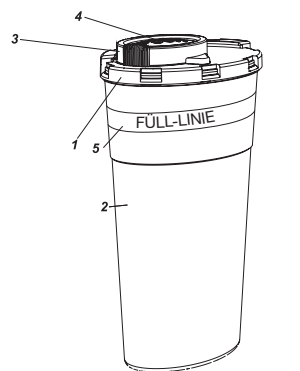




EN Instructions for Use: Multi-Safe *sani*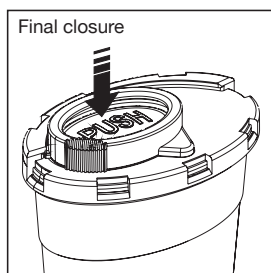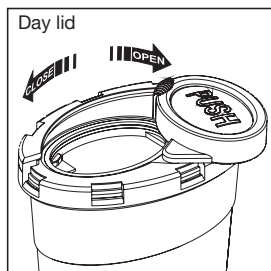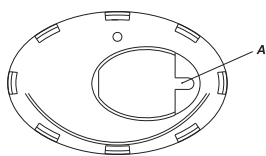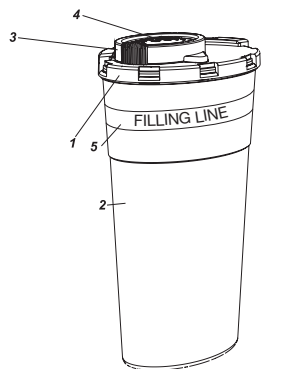

1. Intended purpose and normal use

The disposal box is exclusively intended for the disposal of pointed and sharp items like, for example, needles and scalpel blades for medical use.

2. Safety instructions

- Use only boxes adequately sized to accommodate the items to be disposed.
- Keep the box beyond the reach of children.
- Ensure that items are placed into the box in a vertical direction only.
- Scalpels must be placed into the box with utmost care. Using too much force or filling other objects on top involves the risk of jamming and damage to the box walls or base.
- Do not use force to press objects into the box.
- Do not fill with liquids.
- Do not fill the box above the maximum filling line (5) (health hazard!).
- Do not touch the inside of the box with your hands or otherwise (health hazard!).
- Do not throw.
Do not shake.
Do not drop.
- Ensure that no objects are jutting out before closing.
- Do not use force to close the box and to close by hand only.
- Do not press the box together from the outside when full.
- Make sure that the final closure (4) "PUSH" is firmly secured prior to disposal of the box.

3. Assembly

There is no need for assembly as the box is supplied assembled and fully functional.

4. Daily use

The box features a slide lid (3) and can therefore be managed with only one hand. Move the slide lid (3) into the OPEN or CLOSE position. The box is now ready for transport and can be reopened (as described above) at any time.

5. Final closure of slide lid (3, 4)

After filling [Warning: Do not fill above the maximum fill line (5)], proceed as follows:
To close, move the final closure (3) to the CLOSE position and press the section marked "PUSH" (4) firmly into the base (2). The closure (4) audibly locks into place.

6. Fill line - maximum filling level (5)

For safety reasons, do not fill the box above the maximum filling level (5). (Warning -health hazard!).

7. Holding fixtures (not applicable)

A holder or adapter for this box is not under consideration.

8. Removal aids

8.1 U-shaped recess (A)

for sharps removal from Luer lock systems

9. Disposal and identification

Disposal boxes are made from ecologically friendly polypropylene. Incineration does not cause emission of any harmful substances. Observe statutory provisions pertaining to disposal!

10. DIN EN ISO 23907

If you use the disposal box in agreement with this directive, you will have to insert the absorbent liner below:

Order number	Packaging pcs. / case
78.684	2,000